



iTero – интрооральный сканер


# 1. Настройка интеграции iTero с лабораторией eXceed (делается только один раз)

MYALIGNTECH

**iTero® OrthoCAD®**

Email:

Password:



Remember username

[Forgot Your Password?](#)

  
Science in Every Smile

[Invisalign Doctor Site >>](#)

**Not yet an Invisalign provider?**

See how other professionals are using Invisalign to satisfy their patients and grow their practices. [>>](#)

## 1.1 Войдите в свой аккаунт “My Aligntech”

[Company ID: [redacted] Search Company ID Save Cancel

[New Ticket](#)

Software options for this company can be added using the [New Inventory Movement](#) page

Name 1: [input] \* Locale: en-US  
Name 2: [input] Email: [input]  
Web Site: [input] Phone: [input]  
Working Hours: [input] Status: Active  
Contact Person: [input] Legacy Code(s): [input]  
Representative: [input] LOB: [input]  
Account Num: [input]

Notes: [input]

Tickets Addresses Phones Emails Types Persons Orders Scanners **Relations** Config Activities Audit

+ New Delete

Company ID	Company Name	Relation Type
8201	Align Technology de Costa Rica SRL	Ortho Office - Ortho Modeling Center
8133	Align Technologies Juarez	Ortho Office - Ortho Scanning Center
8133	Align Technologies Juarez	Ortho Office - iQ Fabrication Center

1.2 На главном экране выберете “Relations” и нажмите “New”

Email: [redacted]

### New Relation

Type: Clinical Office - Lab \*

Company: [redacted] \*

OK Cancel

### Search Company

Please select a company by clicking it.

ID	Name	Types	Software Option
14146	eXceed	Lab	Ortho Lab

Page: 1 Go Page 1 of 1 (Total 1 record)

1.3 Чтобы добавить новый вид лаборатории, найдите “eXceed” и выберете его (номер 14146)

**[Company**  Search  Save Cancel

[New Ticket](#)

Name 1:  \*      Locale:  ▼

Name 2:       Email:

Web Site:       Phone:

Working Hours:       Status:

Contact Person:       Legacy Code(s):

Representative:       LOB:

Account Num:

Notes:

---


Tickets   Addresses   Phones   Emails   Types   Persons   Orders   Scanners   **Relations**   Config   Activities   Audit

+ New   Delete

Company ID	Company Name	Relation Type
8201	Align Technology de Costa Rica SRL	Ortho Office - Ortho Modeling Center
8133	Align Technologies Juarez	Ortho Office - Ortho Scanning Center
8133	Align Technologies Juarez	Ortho Office - IQ Fabrication Center
14146	eXceed	Clinical Office - Lab

1.4 Теперь убедитесь, что «eXceed» добавлен в список ваших лабораторий “Relations”

## 2. Отправка сканированных файлов iTero в eXceed (делается в каждом заказе)

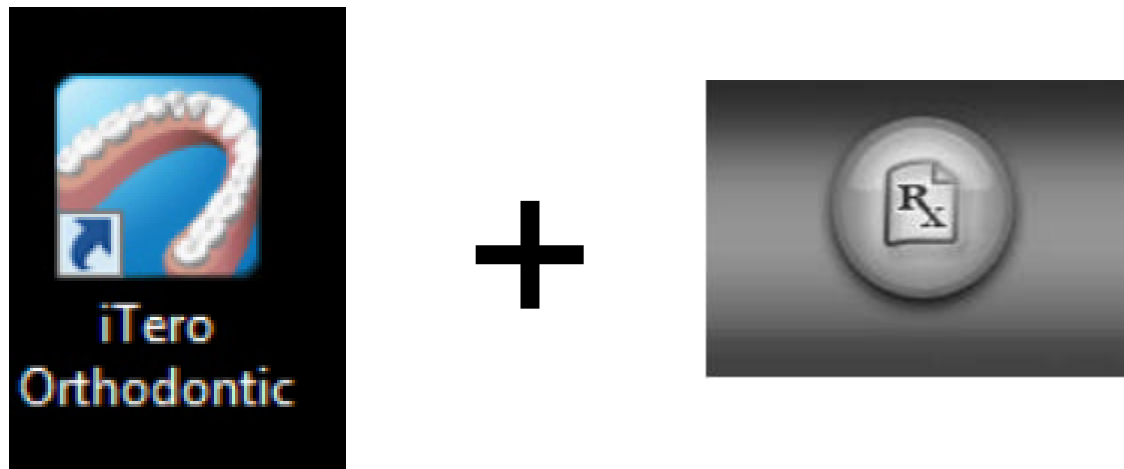

2.1 На сканнере откройте приложение iTero Orthodontic и начинайте заполнение формы (RX)

**Treatment Information**

**Case**

\* Patient First Name: April

\* Patient Last Name: Altman

Chart #:

\* Case Type: iRecord

**Scanning Order**

\* Scanning Order: Lower Jaw First

**Dentist**

\* Name: Mr. Test, TestEamil

\* License #: 123456

Signature:

**Ship To**





Lab: eXceed

To send to a lab, select the lab in "Ship To".

Scan Now    Cancel    Scan Later


2.2 В RX форме, заполните данные пациента, выберете тип кейса iRecord и укажите лабораторию “eXceed” в качестве получателя файлов

### 3. Добавление данных пациента из iTero в личный кабинет eXceed (делается при каждом заказе)

ID	Code	Patient Name
=		
 8365367	91TVJV	Altman, April
 8161999	8UZ79V	
 8126432	8TT6ZN	
 6945106	7H3YPR	
 6935533	7GTZWF	
 6931401	7GNPK6	

3.1 Зайдите в свой аккаунт “My Aligntech” and нажмите ПКМ, чтобы скопировать 7-ми значный id скана



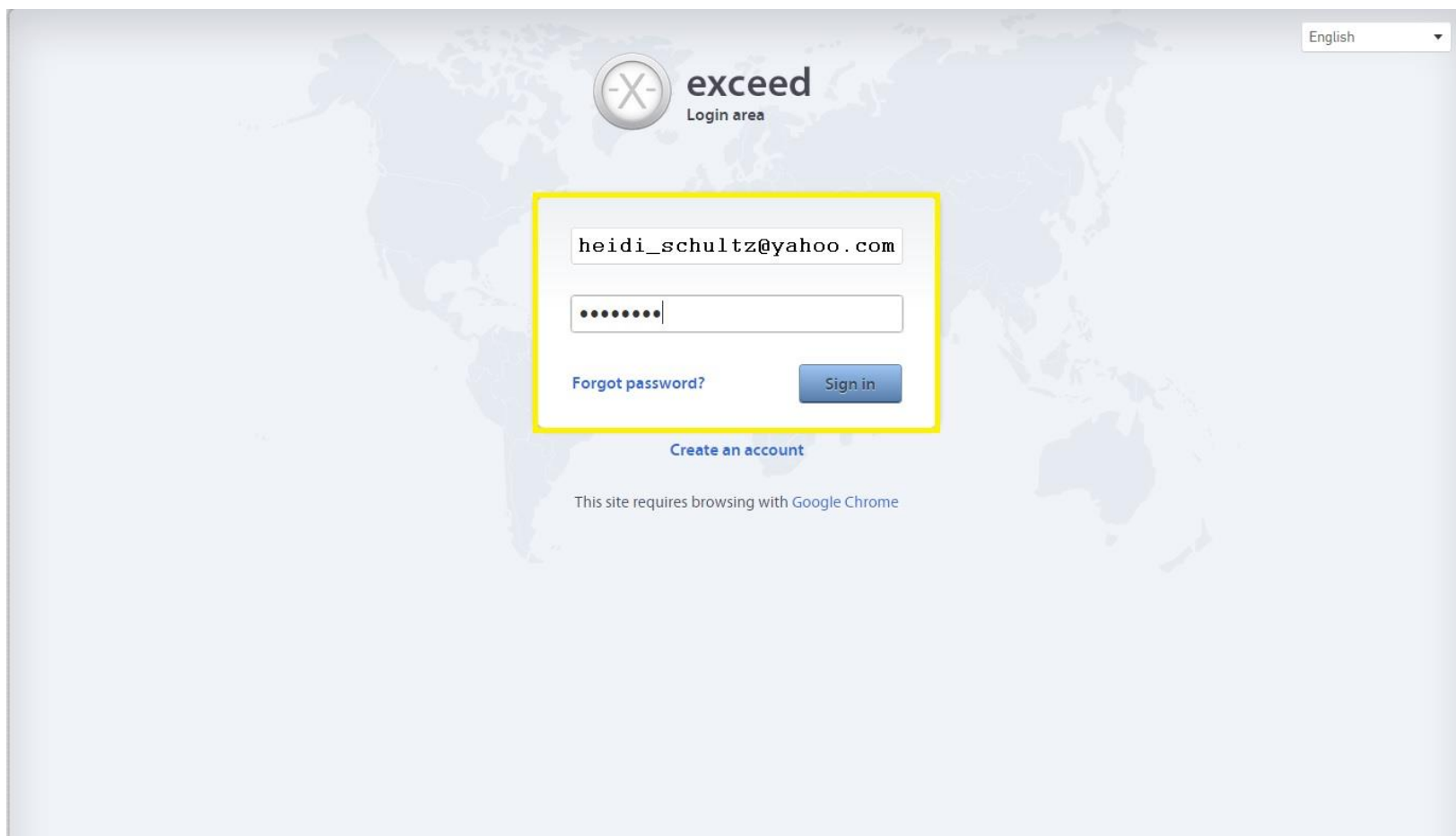

3.2 Войдите в личный кабинет “Му eXceed”, чтобы разместить заказ



3.3 Во вкладке «Данные» нажмите «Выбрать» и вставьте 7-ми значный id скана iTero, После чего необходимо нажать “Enter”. После чего файлы автоматически подгрузятся из iTero в личный кабинет к вашему заказу