



Сканер 3Shape Trios

1.1 – Соединение Trios с eXceed

English

3shape

3shapedental.com

3shape communicate
"For a Full Digital Process"

3Shape
Communicate
Log In

Please enter your email address and password. Register if you don't have an account.

UserEmail heidi_schultz@yahoo.com

Password

Remember me?

[I forgot my password](#)


Log On

Copyright © 3Shape A/S 2011 • info@3shape.com

3Shape A/S • Holmens Kanal 7,4. 1060 Copenhagen K Denmark • P: +45 7027 2620

Войдите в свой 3Shape аккаунт

1.2 – Соединение Trios с eXceed

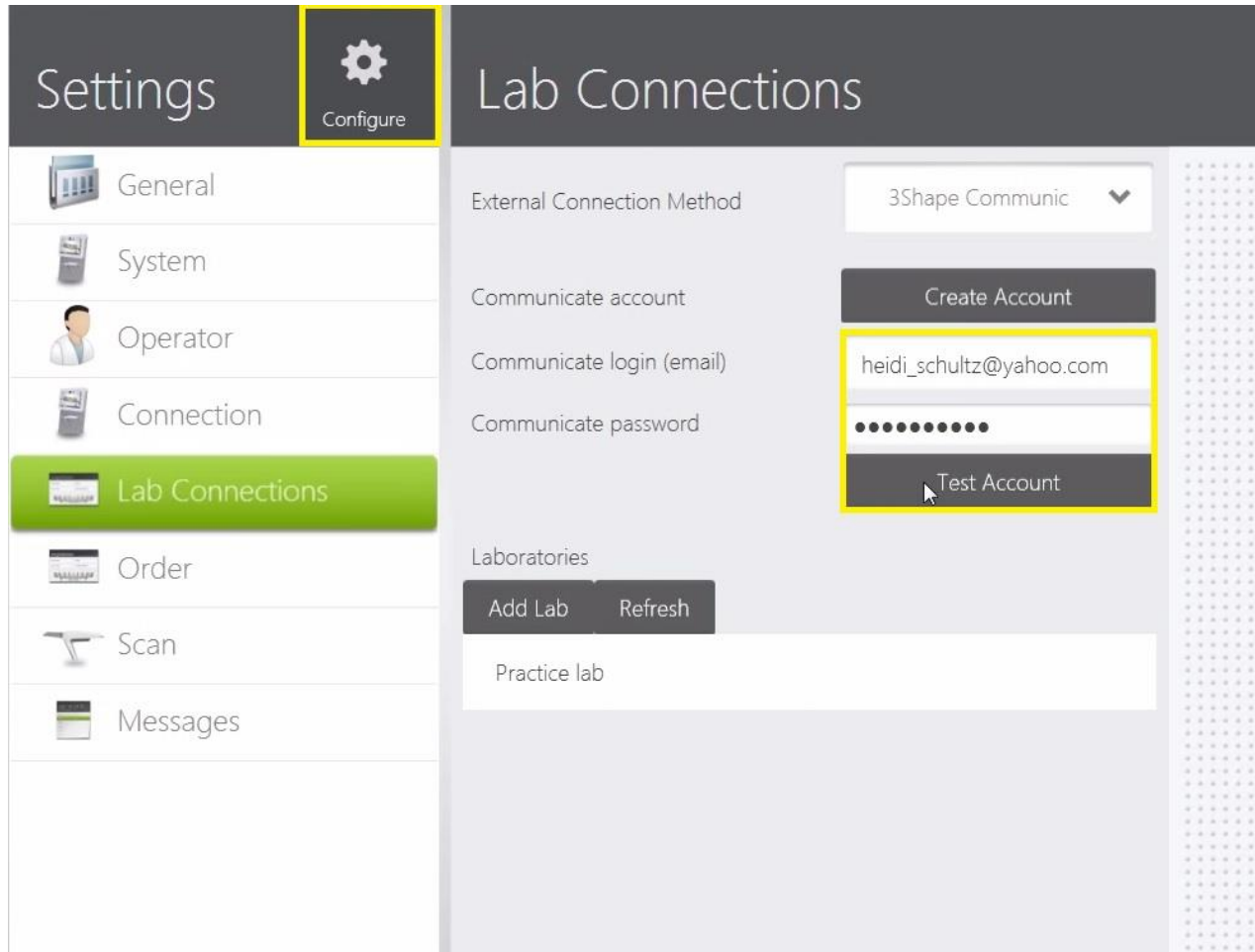

В конфигурации/настройках, подключите тестовую учетную запись


1.3 – Соединение Trios с eXceed

The screenshot displays the 'Settings' application with the 'Lab Connections' section selected. The interface includes a sidebar with menu items: General, System, Operator, Connection, Lab Connections (highlighted), Order, Scan, and Messages. The main content area is titled 'Lab Connections' and contains the following elements:

- External Connection Method:** A dropdown menu set to '3Shape Communic'.
- Communicate account:** A button labeled 'Create Account'.
- Communicate login (email):** A text input field containing 'heidi_schultz@yahoo.com'.
- Communicate password:** A password input field with masked characters (dots).
- Test Account:** A button labeled 'Test Account'.
- Laboratories:** A section with two buttons: 'Add Lab' (highlighted with a yellow box) and 'Refresh'.
- Lab List:** A list box containing one entry: '3Shape Official Demo Lab'.

В меню “Lab Connections”, нажмите добавить лабораторию “Add Lab”

1.4 – Соединение Trios с eXceed

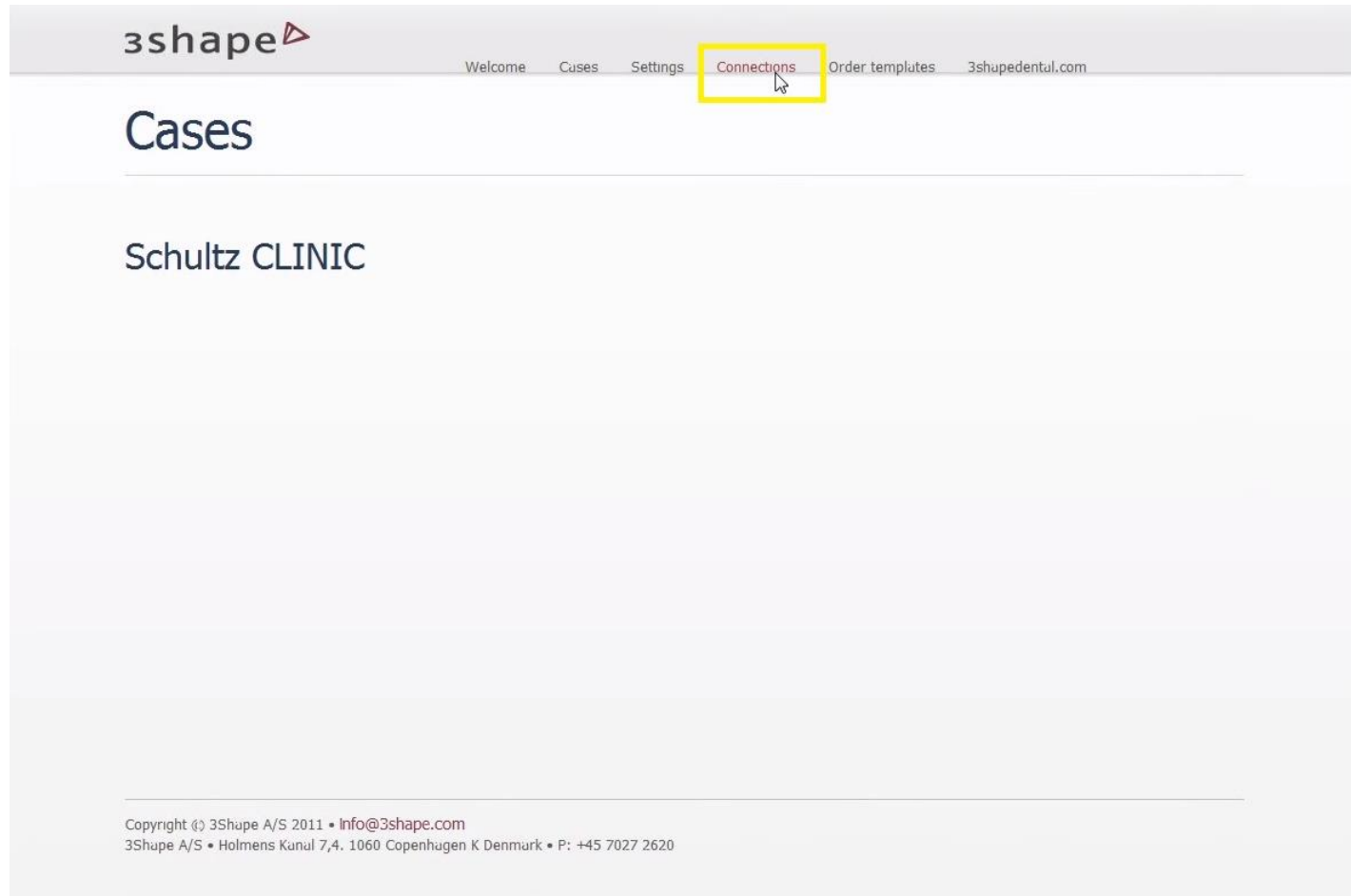

Нажмите подключения “Connections”

1.5 – Соединение Trios с eXceed

Connected Sites


Name


Status

Add Connection

Нажмите добавить подключение “Add Connection”

1.6 – Соединение Trios с eXceed

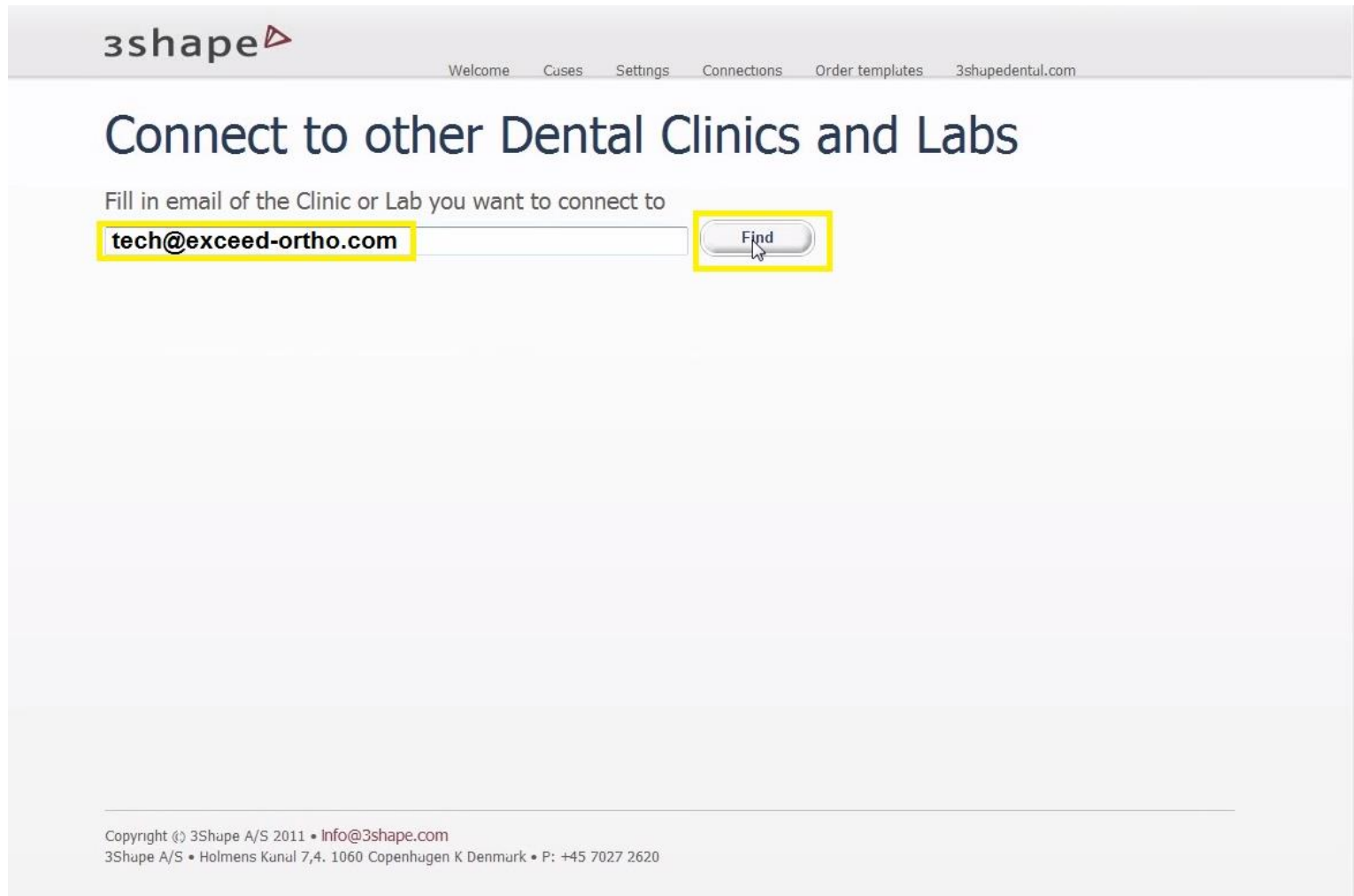

3shape 

Welcome Cuses Settings Connections Order templates 3shapedental.com

Connect to other Dental Clinics and Labs

Fill in email of the Clinic or Lab you want to connect to

Copyright © 3Shape A/S 2011 • Info@3shape.com
3Shape A/S • Holmens Kanal 7,4. 1060 Copenhagen K Denmark • P: +45 7027 2620


Введите `tech@exceed-ortho.com` и нажмите найти “Find”

1.7 – Соединение Trios с eXceed

The screenshot shows the 3shape website interface. At the top left is the 3shape logo. A navigation bar contains links for Welcome, Cases, Settings, Connections, Order templates, and 3shapedental.com. The main heading is "Connect to other Dental Clinics and Labs". Below this, there is a form with the instruction "Fill in email of the Clinic or Lab you want to connect to". The email field contains "tech@exceed-ortho.com" and is highlighted with a yellow box. To its right is a "Find" button. Below the email field, the "Result:" section shows "eXceed" in a box, also highlighted with a yellow box. Underneath is a text area for a message to the lab, with the text "Message to Jasper LAB (You can customize the message)" and "We would like to connect to you on 3Shape Communicate." Below the message area is a "Connect" button, highlighted with a yellow box. At the bottom of the page, there is a footer with copyright information: "Copyright © 3Shape A/S 2011 • Info@3shape.com", "3Shape A/S • Holmens Kanal 7,4. 1060 Copenhagen K Denmark • P: +45 7027 2620".


После того как система обнаружила eXceed – нажмите соединиться “Connect”

2.1 – Подтверждение Trios с eXceed

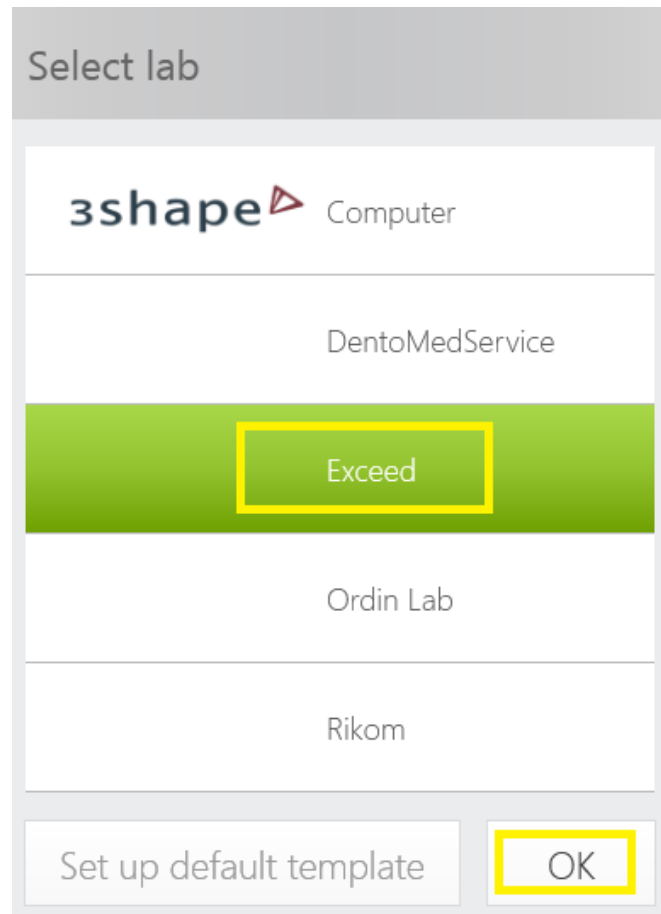


В интерфейсе Trios, выберите “eXceed” в списке лабораторий

2.2 – Подтверждение Trios с eXceed

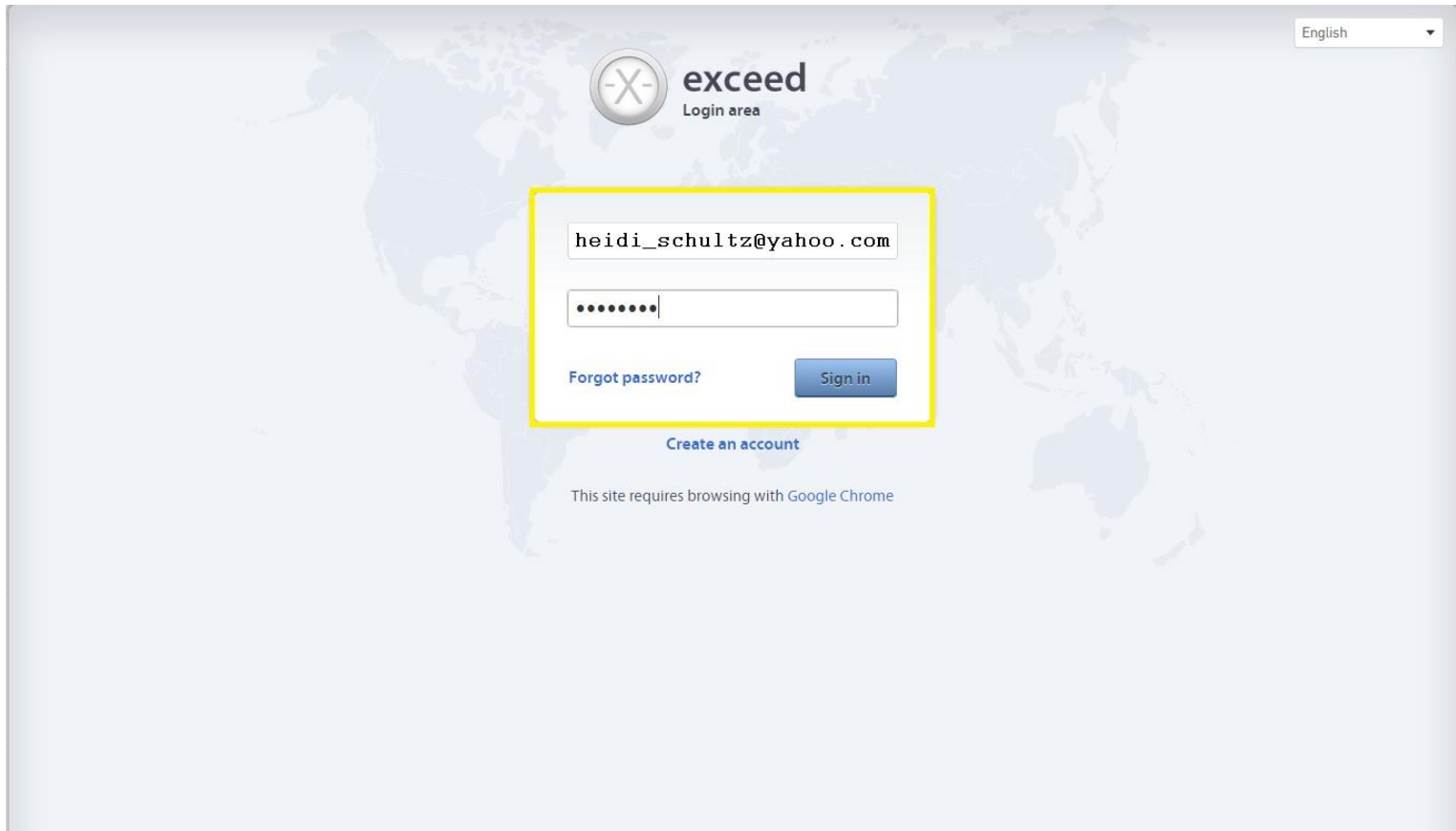

English

 **exceed**
Login area

[Forgot password?](#)

[Create an account](#)

This site requires browsing with Google Chrome

Войдите в свой личный кабинет My eXceed

2.3 – Подтверждение Trios с eXceed

The screenshot displays the eXceed software interface for a patient named Altman April. The 'Models' section is active, showing a 'Patient ID' field with the value '12345678 + hit "Enter"'. A yellow box highlights the 'Select' button, with a callout box containing the text: 'Enable access to models from iTero/Trios intra-oral scanners'. Other callouts include 'Add to order' and 'Add patient's models, For dental impressions or enter the MyAligntech patient ID below. To enable the Lab automatically retrieve the digital models from your scanner, please input patient ID below.' The interface also shows a 'Continue' button and a 'Place a new order' button.

Во вкладке «Данные» нажмите «Выбрать» и вставьте id скана Trios, После чего необходимо нажать “Enter”. После чего файлы автоматически подгрузятся из Trios в личный кабинет к вашему заказу